



Warm Medium

Medium Brown

Warm Dark

Dark Brown

Extra Dark

Intense Dark

DEEP Pigments Best Practices

Our recommendations for the best results with amiea DEEP pigments:

- Shake very well (~ 60 s) before using
- Work in a 90° angle
- Machine stroke: 3.0 or 3.5 mm
- Machine speed: Slow to medium speed
- Needle recommendation: long taper needles, 0.25 or 0.30 mm
- Applied pressure: Lighter touch, avoid high pressure
- Number of passes: 2 to 3
- Diluting the color: Not needed
- Recommended skin types: Thick skin, skin with big pores, oily skin
- Not recommended skin types: Very thin skin, mature skin
- Techniques: Shading, shading between hair strokes, and eyeliner.
- Cleaning: Clean the skin as often as possible to have a better visibility of results; rinse the cartridge or clean the tip

Basic training with organic pigments is recommended